

## LIST OF COMPONENTS:

Resealable plastic bag – 1 L  
Hazpak Medi-Pouch

Absorbent crystal pads

## ASSEMBLY OF PACKAGING:

Remove the narrow section of the tamper evident backing strip ONLY and attach the tamper evident tape to the Medi-Pouch above the opening (The remaining tape should cover the opening with the wider section of the backing strip still in tact). Insert 4 x absorbent crystal pads and re-sealable plastic bag are placed inside the 'Hazpak' Medi-pouch (DO NOT SEAL).

## PRIMARY RECEPTACLE:

<b>Product Code:</b>	IP5-1L	<b>Qty:</b>	1
<b>Description:</b>	Resealable plastic bag.		
<b>Dimensions:</b>	230 (W) x 305 (L) mm x 75 um		

## SECONDARY PACKAGE:

<b>Product Code:</b>	IP5-2L	<b>Qty:</b>	1
<b>Description:</b>	Hazpak Medi-Pouch		
<b>Dimensions:</b>	119 (Dia) x 230 (H) mm	<b>Internal Diameter:</b>	85 mm

## ABSORBENT MATERIAL

<b>Product Code:</b>	AP-CRYSTAL-70x60	<b>Material:</b>	Crystals
<b>Description:</b>	Absorbent crystals inside sachet.		
<b>Dimensions</b>	70 (L) x 60 (W) mm	<b>Qty:</b>	4



**Components**